

## **NAILSEA SHEDDERS DEFIB POLICY    NSP010**

This Policy is the responsibility of the Defib coordinator, currently Phill Wheatley, the person who is acting on behalf of the Trustees for the Charity Nailsea Shedders, and who forms part of the Management Team.

The Policy is in three parts.

Section ONE - About the machine itself

Section TWO – How to Use, Maintain it, then make a record of what you have done.

Section THREE – Rota - See NSP011

### **Section ONE - About the machine itself**

#### Location address:

Wraxall Village Club, Bristol Road, Wraxall, Somerset, BS48 1LE

#### Location description:

On external left-hand wall under the covered porch outside the entrance doors

#### Availability:

[24/7](#)

#### Serial number:

400131807

#### Installation date:

Monday, March 11, 2024, **Battery replacement date: 31 Mar 2028**

#### **Main electrodes / pads expiry date:**

30 Apr 2026 Spare electrodes / **pads expiry date: 30 Apr 2026**

Support type: Basic Support expiry date: Saturday, February 15, 2025

The defib is retained within a lockable heated cabinet. To check the cabinet and to open it, the current code is **123x** Members of the Public who need access, can be told this information by the emergency services.

**Section TWO – How to Use, maintain, then make a record on the Community heartbeat Trust WEBNOS Dashboard.**

Please click on the following link to see a demonstration, how to use the Defib <https://www.defibtech.com/products/lifeline-view-aed/demo/> The Buttons on the video are active, you press them to see what happens next,

***To run a test on the defib, there are three vertical buttons on the right-hand side of the Unit. YOU ONLY NEED TO PRESS THE MIDDLE BUTTON,*** and it will run a self-test. The previous video will show you what you can expect to see, when you press the middle of the three vertical buttons on the right-hand side of the Unit.

Once you have checked it, Each Defib checker has a unique login and password to get into the WEBNOS ambulant compliant section. Once you have completed your checks **EMAIL THE NEXT PERSON WITH THE DATE, THEY NEED TO CHECK IT BY, AS WHEN YOU COMPLETE YOUR CHECKS, THE DATE STAMP AUTOMATICALLY DEFAULTS TO THE NEXT MONTH.**

Before you can access your WEBNOS dashboard, you need to have a login name and password. This you can get from the Community Heart Beat Trust directly through their website <https://www.communityheartbeat.org.uk/> If you have problems with this, get in touch with the Defib Coordinator for further assistance.

Once you have your login and password, once you have completed the Defib checks, you access your dashboard through the following portal.

<https://www.defibcheck.org/user/login?destination=node/3>